

AUTOMATIC GAS AMPOULE BREAK ACTIVATOR



PRODUCT FEATURES

ZE OSS automatic lamp activator provides automatic activation of the gas lamp used in ethylene oxide sterilizers operating with gas diffusion technique, without touching the hand. Press the power button. After pressing the power button, check the charge status of the device by pressing the control button. Make sure that the device is not fully discharged. Follow the led lights by pressing the control button of the device.

When the yellow LED light turns on, leave the device on charge for 6 hours. When the charge is full, the LED light will be green.

The red LED lamp indicates that the battery is completely dead. Running out of battery will shorten battery life. It is recommended to use the device in accordance with the specified standards for a healthy and long-lasting use.

CODE	PRODUCT NAME	EXTERNAL DIMENSIONS (MM)
ZEO-ACT	ETHYLENE OXIDE AMPOULE AUTOMATIC BREAK ACTIVATOR	185x60x160mm Voltage: 220