

DUAL INCUBATOR



PRODUCT FEATURES

Product Description

Code: ZDL-100

Incubator for long-term biological incubator.

- Dimensions: 15 x 13 x 9 cm (width x length x height)
- Weight: 1,7 kg
- Chrome plated
- 29 metal slots for incubation (0,9 cm diameter, 3,0 cm depth)
- 4 metal slots for incubation (1,3 cm diameter, 3,0 cm depth)
- 1 bulb breaker.
- Digital display
- Self-timer
- Alarm

Use

- Plug the device into the socket.
- Select the mode you want to use by pressing the button.
- After selecting the mode, the timer will start automatically and the device will warm up to the required temperature.
- Break the biological indicator ampoule with the help of the ampoule breaker, then place the biological indicator in the appropriate slot for incubation.
- The alarm will sound when the incubation period is over. Press the "reset" button to turn off the alarm 6-You will get the results at the end of the incubation period.
- To change the mode; After pressing the "reset" button, select the mode you want to use.
 - MOD 1: 37 ± 2 °C – 48 hours (Ethylene Oxide)
 - MOD 2: 57 ± 2 °C – 24 hours (Steam)
 - MOD 3: 60 ± 2 °C – 24 hours (Hydrogen Peroxide)

Warning

- Do not spill any liquid into the device.
- Do not immerse the device in any liquid.
- The device is for indoor use only.
- Unplug the power cord before cleaning the device.
- Do not use abrasive or abrasive cleaners or disinfectants for device cleaning.
- Make sure the device is connected to a suitable power cord