

## ETHYLENE OXIDE GAS NEUTRALIZER



### PRODUCT FEATURES

#### PRODUCT CODE: ZEISS NTR

ZEISS Neutralizer transforms Ethylene Oxide gas into Ethylene Glycol with its highly effective neutralization effect, making it harmless to the environment.

### INSTRUCTIONS FOR USE OF NEUTRALIZER

- Fix the neutralizer wheels.
- Fill the neutralizer acid tank with acid up to the marked place on the side of the acid level indicator pipe.
- Fill the neutralizer water tank with water up to the marked place on the side of the water level indicator pipe.
- Minimum acid and water levels, neutralizer acid and water level indicator pipes are shown on the side. **When acid or water reaches a minimum level, it should be refilled.**
- Completely empty and refill the water tank once every 3-6 months. The water drain valve is the valve at the bottom of the water level indicator hose.
- To fill the water tank, connect the hose on the water level indicator pipe to the tap and fill the tank.